

THE HYGIENIC RUBBER CHECK PESSARIE

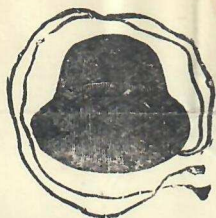


Fig 1.

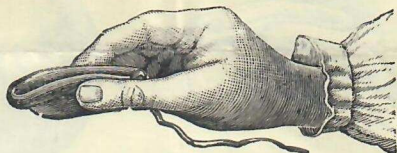


Fig. 2.

EXPLANATION OF THE ENGRAVINGS.

Fig. 1 is the Check Pessarie, a simply devised instrument of pure soft medicated rubber, to be worn by the female for supporting and covering the womb. It is constructed on a common sense principle and strictly in accordance with the female organisation, can be worn any length of time with ease and comfort; and so accurately fits the parts that it adapts itself perfectly and no apprehension of its going too far or doing the slightest harm need be felt. With care it will last for years. It is recommended by medical men to delicate females who, by distortion of the pelvis cannot with safety bear children.

DIRECTIONS FOR USE.

Damp the PESSARIE with soap suds, double it with the hollow surface inwards (see Fig. 2), then introduce it with the hollow surface upwards until it meets with an obstruction, when it will open; and then with a little pressure with the fore finger around the ring, it will adapt itself to the female organs so as to cover the mouth of the womb, and thereby prevent anything from entering. As this is the acting principle on which success of the instrument depends, great care should be taken that it is properly adjusted, which can be found out by feeling a firm substance in the centre or soft part of the PESSARIE, which will be the productive organ.

It can be easily removed in a stooping position (the same as it is introduced) by lifting the lower edge of the rim with the fore finger, but as fear may attend in some cases on the first using an improved silken arrangement is attached by which means it can be used and ejected with perfect safety and without fear; and the silken arrangement can be disconnected when the wearer is confident to use it without. It should not be removed until six hours after use, but within twenty four, then washed with soap and water; replaced in box, and kept in a dry, warm place. If you have a Vaginal Syringe (the Hygienic Whirling Spray Syringe is the best) it is always best to wash out the passage after removing the PESSARIE. This is not obligatory.

THE HYGIENIC STORES LIMITED,

95, Charing Cross Road, London, W.C.2.

The Hygienic "Anti-Geniture"



Fig. 1

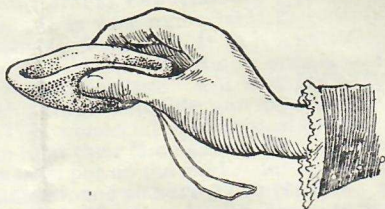


Fig. 2

EXPLANATION OF THE ENGRAVINGS.

Fig. 1 is the ANTI-GENITURE, a simply devised instrument of pure soft medicated rubber, to be worn by the female for supporting and covering the womb. It is constructed on a common-sense principle and strictly in accordance with the female organisation; can be worn any length of time with ease and comfort; and so accurately fits the parts that it adapts itself perfectly and no apprehension of its going too far or doing the slightest harm need be felt. With care it will last for years. It is recommended by medical men to delicate females who, by distortion of the pelvis cannot with safety bear children.

DIRECTIONS FOR USE.

Damp the ANTI-GENITURE with soap suds, or preferably Hygienic Quinine Compound, double it with the hollow surface inwards (see Fig. 2), then introduce it with the hollow surface upwards until it meets with an obstruction, when it will open; and then with a little pressure with the fore-finger around the ring, it will adapt itself to the female organs so as to cover the mouth of the womb, and thereby prevent anything from entering. As this is the acting principle on which success of the instrument depends, great care should be taken that it is properly adjusted, which can be found out by feeling a firm substance in the centre or soft part of the ANTI-GENITURE, which will be the productive organ.

The ANTI-GENITURE can be easily removed in a stooping position (the same as it is introduced) by lifting the lower edge of the rim with the fore-finger, but as fear may attend in some cases on the first using an improved silken arrangement is attached by which means it can be used and ejected with perfect safety, and without fear; and the silken arrangement can be disconnected when the wearer is confident to use it without. It should not be removed until six hours after use, but within twenty-four, then washed with soap and water, replaced in box, and kept in a dry, warm place. If you have a Vaginal Syringe (the Hygienic Whirling Spray is the best, it is always best to wash out the passage, after removing the ANTI-GENITURE, for this purpose a Hygienic Spray Antiseptic Tablet should be used (see pages 8 and 9 of catalogue).

THE HYGIENIC STORES, LTD.,
95 Charing Cross Road, London, W.C. 2.

HYGIENIC

Soluble "Quinine Compound"

(TREBLE STRENGTH)

For use with the Rubber Safety Sponge or other safety sponges and check pessaries. The use of the Quinine Soluble Compound is not absolutely necessary, but on account of its germ destroying properties, is strongly recommended; it makes the action of the sponge certain. We advise all users of sponges to use this compound, it being an additional safeguard, thus making **failure impossible**. It is also advisable to use it with the "Check Pessary," a thin coating being applied to the rim of the pessary which makes it doubly sure. The compound is made in **treble strength** only, and is quite harmless. Its action is the same as the Quinine Pessary.

Small box 1/- each ; Large box 1/6 each ; Largest 2/6. Post Free

95 CHARING CROSS ROAD, LONDON, W.C.2