


Hygienic Whirling Spray Syringe

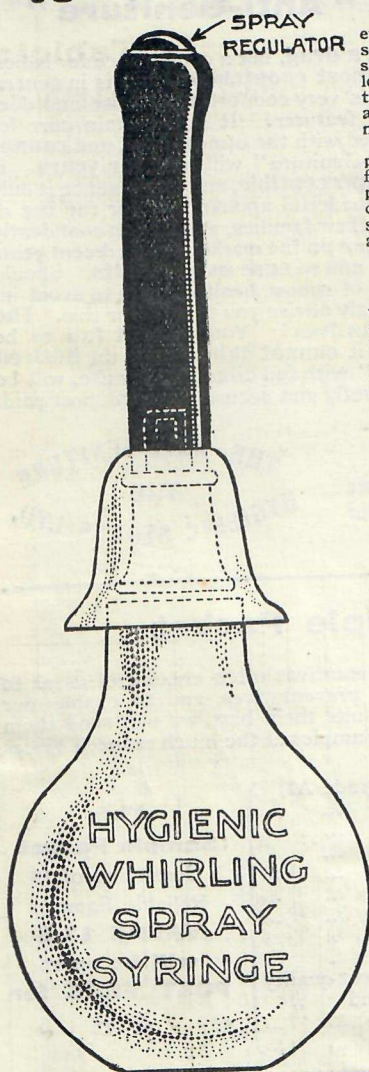

The most wonderful improvement ever made in injection and suction syringes. It is entirely unlike the old style clumsy solid-stream enemas, so long found inefficient and unsatisfactory. It is most simple; convenient and effective. Always ready for use—nothing to get out of order.

The syringe dilates and flushes the passage with a volume of whirling fluid which smooths out the folds, and permits the injection to come into contact with its entire surface, instantly dissolving and washing away all secretions and discharges.

HOW TO USE THE SYRINGE

Place the nozzle of the syringe in a vessel of antiseptic fluid with bulb compressed, release pressure on bulb until the syringe is filled. It can be used lying down with as great, if not greater facility than in any other position, but a convenient sitting position will answer as well. By relaxing the pressure, the injection, together with all secretions (dissolved) will be removed.

Re-inject a number of times until satisfied that the sticky mucous lining of the parts has been thoroughly washed out.

It will be seen from this brief description how simple and convenient it is. There has been nothing introduced up to the present that does the work so effectively as THE HYGIENIC WHIRLING SPRAY SYRINGE.

Made in Five Qualities—

5/6, 8/6, 12/6, 15/6 & 21/- Post Free.

Unobtainable Elsewhere.

Full Directions Enclosed.

Recommended by London's Greatest Physicians.

See next page for Antiseptic Tablets for use with Syringe.

Supplied only by

THE

HYGIENIC STORES LTD.,

Surgical Appliance Makers,

95 Charing Cross Road,
London, W.C. 2.

Telephone: GERRARD 6123.

Telegraphic Address:

"Hygistor, London."