


DILA-SPRAY

VAGINAL SYRINGE

*No
Complete
Irrigation
Without
Dilatation*

Copyright 1927 by CONTINENTAL LABORATORIES, INC.

CONTINENTAL LABORATORIES, INC.
100 BEEKMAN ST. NEW YORK, N. Y.

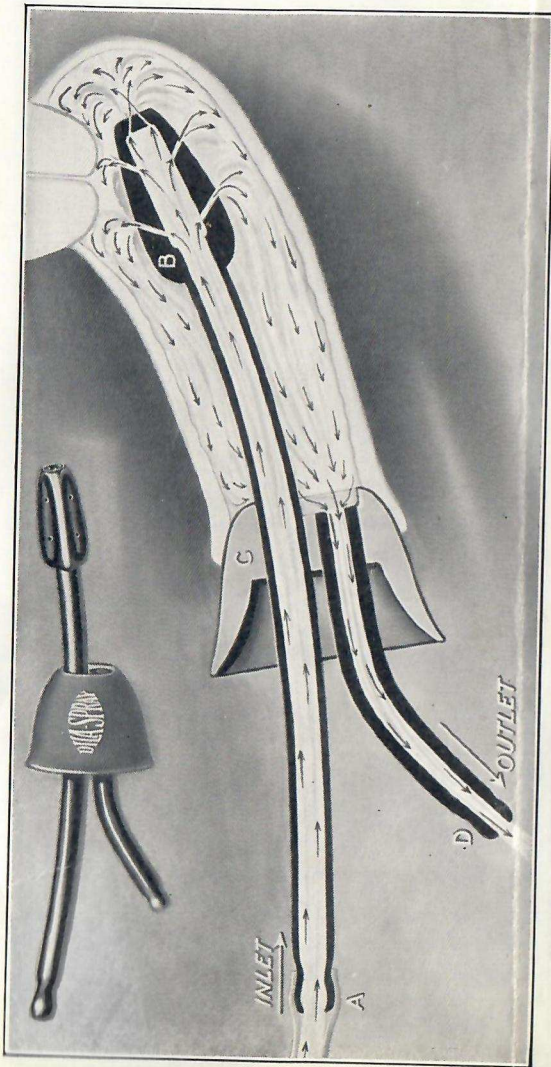

DILA-SPRAY Vaginal Syringe (Patented October 26, 1926)

The DILA-SPRAY, a new vaginal syringe with return flow, is designed to produce a ***prompt, safe and thorough irrigation*** of the vaginal canal.

The fluid, entering from the douche bag through the inlet tube A with the nozzle B, irrigates from the inside outward. The nozzle B is provided with a number of large openings in the grooves on the side as well as with 3 small apertures in front. The latter are directed toward the axis of the inflow tube A at an angle of 45°, a new special design to insure ***irrigation around the orifice of the womb*** without injuring it by a direct, solid and violent stream or other infectious germs, accumulated secretions or other undesirable discharges upward into the cervical canal. The oval soft rubber flange C, held in the entrance to the vaginal canal, directs the return flow through the outlet D.

The novel and most important feature of the DILA-SPRAY is that the combined areas of the inflow openings are considerably larger than the diameter of the outlet tube (see illustration). By this new design, the volume of the inflow is automatically so regulated that it is continually larger than the outflow. Through the pressure resulting from the excess inflow, a ***gradual and gentle but complete and continued dilatation*** or distention ("ballooning") of the vaginal walls is produced smoothing out the numerous natural folds and pockets, the favorite hiding places of germs and impurities. In this way, the douching liquid is given easy access to every spot and can thoroughly cleanse, deodorize or disinfect the ***entire surface*** of the vaginal tract by a ***continuous irrigation***.